

Guide for PortaCount™ Data for TSI Link™ for Respiratory Protection



Background

Ensuring clarity about what fields represent and how they are displayed in the system helps to ensure you import the data properly and understand what is shown in the Portal.

This guide provide links to a examples file, to an import template and provides a data dictionary. This can help you understand the PortaCount™ Data for TSI Link™ for Respiratory Protection.

Links to Resources

[PortaCount Import Template](#)

[PortaCount Import Example](#)

[TSI Link for Respiratory Protection API documentation](#) (Login Required)

Data Dictionary

PortaCount™ Fields

Field Name	TSI Link field Name*	Required	Field Type	Example	Notes
name	Name	No	toText(64)	Building 270 PortaCount	Manually Input
serialNumber	Serial Number	Yes	toText(64)	8048874632	Manually input, should be 10 numbers long
model	Model	No	toText(64)	8048	Automatically populated based on Serial Number
lastCalibration	Last Calibration	No	toISODate	2024-08-26T18:40:43.355Z	Date in the past
location	Location	No	toText(64)	Room 5	
planInfo	Plan Information	No	toText(64)	B2B5 Plan, will need to purchase a CLC Plan	Open Text describing service plan, up to 128 characters
planExpiration	Plan Expiration	No	toISODate	2024-08-26T18:40:43.355Z	Manually input

Note * *Italics* = not a specific field on the TSI Link website, so the use is described



Knowledge Beyond Measure.

TSI Incorporated – Visit our website www.tsi.com for more information.

USA Tel: +1 800 680 1220
UK Tel: +44 149 4 459200
France Tel: +33 4 91 11 87 64
Germany Tel: +49 241 523030

India Tel: +91 80 67877200
China Tel: +86 10 8251 6588
Singapore Tel: +65 6595 6388

TSI and the TSI logo are registered trademarks of TSI Incorporated in the United States and may be protected under other country's trademark registrations.



5003306



Distributed by: Kenelec Scientific Pty Ltd | 1300 73 22 33 | sales@kenelec.com.au | www.kenelec.com.au